HEEL LIFT CUSHIONS with Physio'Choc™

INDICATIONS

Prevent and alleviate joint pain and back pain caused by shock waves that pass through your body at each step. Relieve pain resulting from calcaneal spurs.

CONTRA-INDICATIONS

- Do not use on a skin lesion.
- Persons suffering from arteritis, diabetes, polyneuritis, lower limb neuropathy or varicose ulcers: never use the product on a wound, monitor the condition of your foot during use, and in all cases seek advice from your doctor or chiroprapist.

PRECAUTIONS FOR USE

- The heel lift cushions should be worn with socks, tights or stockings.
- Before using a cushion for the first time, remove the protective film covering the double-sided adhesive. Position the heel lift cushions in your shoes.
- If the heel lift cushions are too wide for your shoes, they can easily be cut to the desired width using a pair of scissors.

WHAT IT IS USED FOR

- Approved by chiropradists, Physio'Choc™ is a powerful shock absorber. Heel lift cushions made from this material help to dissipate the energy created by foot/ground contact: consequently, joint pain and back pain are alleviated.
- In addition, by raising the heel by 5mm, Heel Lift Cushions reduce tension in the tendons and provide appreciable comfort to people suffering from calcaneal spur.

CLEANING

- The heel lift cushions are not washable.
- Discard and replace the device as soon as tears in the heel lift or cracks in the silicone pad appear.

COMPOSITION 76% Physio'Choc™ (Elastomer), 24 % silicone.

In case of pain or any other unusual reaction, contact your doctor. Any major incidents involving the device must be reported to the manufacturer and the relevant health authority. Millet Innovation cannot be held responsible for failure to use the product in accordance with the manufacturer's recommendations. French is the reference language. CE initial marking: 2005.

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