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Ref: 10944-H_06/2023

CALLUSE PROTECTIONS with EPITHELIUM™ Activ

INDICATIONS

To protect the skin and provide pain relief from hard skin and calluses on the front of the foot etc.

CONTRAINDICATIONS

- Do not use on a skin lesion.
- In the event of an allergic reaction, or substantial irritation, the treatment should be stopped.

PRECAUTIONS FOR USE

- Protections should be applied to clean, dry skin.
- Remove a protection from the metallic envelope.
 - Remove the protective film covering the adhesive side.
 - Apply the adhesive surface to the skin, making sure that the affected area is well covered.
 - Apply light pressure to the protection to ensure that it has adhered to the foot.
 - Gently cover with socks, stockings or tights.
 - Keep the protection in place during the day.
 - Remove at night, place on the tray provided for this purpose and return to the metallic envelope until the next day.

For optimum results it is recommended that the protection is applied everyday.

WHAT IT IS USED FOR

- The effectiveness of these self-adhesive protections is **due to EPITHELIUM™ Activ gel which evenly distributes pressure: walking becomes more comfortable as soon as they are applied.**
- Very effective, and yet **very thin**, the protections may be worn in all types of shoes (formal, sports, house etc.).

These protections have been tested and approved for use by diabetics.

CLEANING

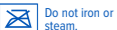
- Wash after use. Pads may be washed with soap and water. Do not rub, allow to air-dry.
- Discard and replace the device as soon as cracks in the gel or fabric damages occur.
- Leave to dry naturally. Do not iron.

COMPOSITION: 84% silicone, 16% polyamide/elastane fabric.

In case of pain or any other unusual reaction, contact your doctor. Any major incidents involving the device must be reported to the manufacturer and the relevant health authority. Millet Innovation cannot be held responsible for failure to use the product in accordance with the manufacturer's recommendations. French is the reference language. CE initial marking: 2006.

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