## TÅBESKYTTELSER MOD BLÅ NEGLE

EPITHELIUMTACT 02

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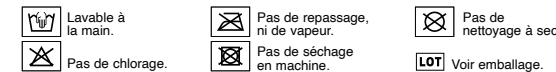
INDICATIONS: Toenail protectors.

## CONTRAINDICATIONS:

- Do not use on an unprotected wound.
- Individuals with arterial disease, diabetes, neuropathy of the lower limbs or varicose ulcers: never use the product over an open wound or diabetic foot ulcer. Monitor the foot during use and ask your doctor or podiatrist for advice.

## PRECAUTIONS FOR USE:

- These protectors should be used **preventively**. They are also suitable for use once a bruised nail has formed.
- The protectors must be used on clean, dry skin. Wash before first use.
- For easy application, position the protector inside-out on the end of the toe, then gradually unroll it.
- Do not shorten the protector since this weakens it.
- Remove the protector at night to avoid any risk of maceration.
- Place the protector in the foil sachet provided until next use.

## WHAT IT IS USED FOR:

EPITHELIUMTACT 02 toenail protectors are made of ultra-resistant silicone gel, of 1 mm thickness only, with visco-elastic properties similar to those of the skin. The protectors thus distribute pressures and absorb micro-shocks by dissipating energy in the gel and thereby prevent the risk of bruised nails or blisters. They also relieve pain, allowing wearers to continue their sports activity, while maintaining their performance.

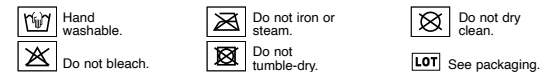
## RECOMMENDED CARE:

- Wash after use. Protections may be washed with soap and water.
- Discard and replace the device as soon as cracks in the gel or fabric or seam damages occur.
- Do not rub, allow to air-dry.

**COMPOSITION:** 82% silicone, 18% polyamide/elastane fabric.

In case of pain or any other unusual reaction, contact your doctor. Any major incidents involving the device must be reported to the manufacturer and the relevant health authority.

MILLET Innovation cannot be held liable in cases where the manufacturer's recommendations have not been followed during use. French is the reference language. CE initial marking: 2014.

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