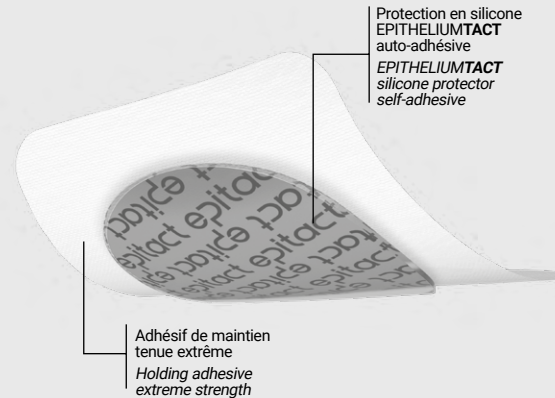
PROTECTIONS ANTI-AMPOULES EPITHELIUMTACT 01

BLISTER PROTECTORS / BLASENSCHUTZ / PROTEZIONI ANTI-VESCICHE / PROTECCIÓN ANTI BOLHAS / BESCHERMERS TEGEN BLAREN / PROTECCIONES ANTIAMPOLLAS / BESKYTTELSER MOD VABLER / SKAVSÅRSKYDD / RAKKOSUOJAT / GNAGSÅRPLASTER / ΠΡΟΣΤΑΣΙΑ ΚΑΤΑ ΤΩΝ ΦΛΥΚΤΑΙΝΩΝ

19378_EL_09/2023

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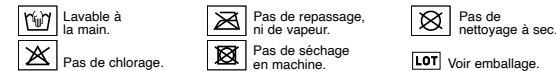
INDICATIONS: Blisters protectors.

CONTRAINDICATIONS:

- Do not use on unprotected wound.
- Individuals with arterial disease, diabetes, polyneuritis, neuropathy of the lower limbs or varicose ulcers: never use the product over an open wound or diabetic foot ulcer. Monitor the foot during use and ask your doctor or podiatrist for advice.

FUNCTIONS:

EPITHELIUMTACT01 blisters protectors are made of self-adhesive silicone gel, of 1 mm thickness only, with visco-elastic properties similar to those of the skin. Applied to sensitive areas of the foot before sport, they distribute pressures and absorb frictions by dissipating energy in the gel and thereby prevent blisters forming. Contact with the skin is very comfortable and the protector peels off painlessly, even when used on hairy areas.

INSTRUCTIONS FOR USE:

These protectors should be used preventively. They are also suitable for use once a blister has formed, as long as it has not burst.

The protectors must be used on clean, dry skin.

- Take a protector out of its foil wrapper.
- Carefully peel off the protective film covering the adhesive side.
- Apply this adhesive side to the skin, being careful to fully cover the area to be protected.
- Press the protector gently to make sure it sticks properly.
- Cover the protector with holding adhesive to ensure impeccable hold (centre the protector, leaving a margin of 2 cm of adhesive around the edge).
- After use, remove the holding adhesive from the skin and carefully peel off the protector.
- Place the protector on the storage carton provided and place in the foil sachet until next use.

WASHING INSTRUCTIONS:

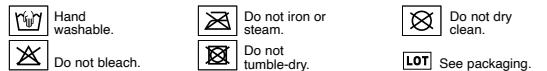
- Wash after use. The protectors can be washed with soap and water 4 to 5 times on average without losing their adhesive properties.
- Discard and replace the device as soon as cracks in the gel or fabric damages occur.
- Do not rub and leave to dry naturally.

COMPOSITION:

- PROTECTOR: 84% silicone, 16% polyamide/elastane fabric.
- HOLDING ADHESIVE: 100% Biocompatible.

In case of pain or any other unusual reaction, contact your doctor. Any major incidents involving the device must be reported to the manufacturer and the relevant health authority.

MILLET Innovation cannot be held liable in cases where the manufacturer's recommendations have not been followed during use. French is the reference language. CE initial marking: 2014.

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