HALLUX VALGUS ORTESE EPITHELIUMFLEX® 02

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COMPOSITION : 52% silicone, 48% tissu polyamide/elasthaane.

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epitact^{SPORT}

HALLUX VALGUS BRACE EPITHELIUMFLEX® 02

INDICATIONS:
This flexible hallux valgus brace helps to realign the big toe, limit pronation and thereby optimise propulsion. Its EPITHELIUMFLEX® external support absorbs pressure and friction on the exostosis («bunion»). Finally, its supportive metatarsal wrap helps to limit crushing and flattening of the forefoot.

- CONTRAINDICATIONS:**
- Big toe not mobile.
 - Do not use on a skin lesion.
 - Individuals with arterial disease, diabetes, polyneuritis, neuropathy of the lower limbs or varicose ulcers: never use the product over an open wound or diabetic foot ulcer. Monitor the foot during use and ask your doctor or podiatrist for advice.

- PRECAUTIONS FOR USE:**
- Wash before first use.
 - First of all, make sure that the big toe can still move sideways when manipulated. Otherwise, do not use this product.
 - Do not use this product at night.
 - The brace can be worn on either the right foot or the left foot.

- FITTING INSTRUCTIONS:**
- Slip the front of the forefoot into the wrap.
 - Then slip the brace over the big toe.
 - Position the Epithelium™ pad (a) behind the bunion (b). (See diagram 1).

- ADJUSTMENT:**
- When using for the first time, do not try to completely realign the big toe.
 - Over the course of the first 10 uses, gradually realign the big toe by pulling the brace backwards. (See diagram 2).

The brace may be worn daily for optimum results.

- WASHING INSTRUCTIONS:**
- Washing after use at 30°C or wash by hand (without rubbing).
 - Discard and replace the device as soon as cracks in the gel or fabric or seam damages occur.
 - Leave to dry naturally. Do not iron.

COMPOSITION: 52% silicone, 48% polyamide/elastane fabric.

In case of pain or any other unusual reaction, contact your doctor. The silicone tendon in contact with the skin may cause mechanical irritations, such as blisters (< 0.1%). Any major incidents involving the device must be reported to the manufacturer and the relevant health authority.

MILLET Innovation cannot be held liable in cases where the manufacturer's recommendations have not been followed during use. French is the reference language. CE initial marking: 2014.

- Machine washable at 30°C.
- Do not iron or steam.
- Do not bleach.
- Do not tumble-dry.
- See packaging.

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