# CHEVILLÈRE LIGAMENTAIRE

LIGAMENT ANKLE BRACE ,  
ENKELBANDAGE GEWRICHTSBANDEN,  
CAVIGLIERA PER LEGAMENTI,  
KNÖCHELBANDAGE

FRANÇAIS

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See packaging.

ENGLISH

**INDICATIONS:** Wearing the EPITACT® ligament ankle brace is recommended for benign, moderate sprains of the lateral collateral ligament and when resuming activity following a moderate or severe sprain. It is indicated for treatment of postoperative oedema.

**CONTRAINDICATIONS:** Do not use in case of diabetes, polyneuritis, neuropathy of the lower limbs and arteritis. Not for use on cutaneous lesions.

**PRECAUTIONS FOR USE:** Wash before first use. Before using the ankle support, read the information in the fitting leaflet. If in doubt, ask your doctor or your pharmacist, or any other competent professional for advice. This medical device is a regulated healthcare product which bears CE marking, according to legislation. Assessment of compliance was performed by the ASQUAL.

**LIFESPAN :** Wash after use. Hand washing at 30°C, with a mild soap, without detergent or bleach. Rinse well. Squeeze to expel moisture. Do not expose while drying to a source of direct heat (radiator, sunlight). Do not use any stain removers. Discard and replace the device as soon as cracks in the gel or fabric or seam damages occur.

**COMPOSITION :** 82,9 % textil (Polyamide, Elasthan, Polyester), 17,1% Silicone.

**FEATURES:** The 3D elastic weave takes perfect account of the volume of the foot and exerts uniform pressure on the joint for enhanced proprioception. It is combined with a patented strapping system that stabilises the ankle and maintains the foot in line during each of your movements without applying any pressure to the Achilles tendon.

**ERGONOMICS:** The detachable strapping allows adjustment of tightness to suit your needs (oedema, etc.). Anatomically shaped, the ligament ankle support features a closed heel and a specific weave at its ends to avoid a tourniquet effect. Its micro-aerated stitching enables the release of sweat for greater comfort. The EPITACT® ligament ankle support can be worn interchangeably on either the right or left ankle.

**TIP:** If you are an athlete, when resuming activity, we recommend you use another product from the EPITACT® range: the ERGOstrap™ Sport to secure your fragile ankle.

In case of pain or any other unusual reaction, contact your doctor. Any major incidents involving the device must be reported to the manufacturer and the relevant health authority. Millet Innovation cannot be held responsible for failure to use the product in accordance with the manufacturer's recommendations. French is the reference language. CE initial marking: 2017.

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